

Quick Start Guide

IntelliGo i3 CGMS + IGO - Share App





- IntelliGo App on your smart phone receives your glucose data from Transmitter and displays your glucose information.
- IntelliGo App does not use internet (**offline**), and your personal information is only stored on your smart phone.

- iGO - Share App on your smart phone receives your glucose data from IntelliGo App and send data to the cloud server.
- iGO - Share App uses internet (**online**), and your personal information will be safely used after your consent to generate analysis reports and graphs for diabetes management.

Data flow: CGM device ➡ IntelliGo App ➡ iGO-Share App ➡ IntelliGo Share Web Portal

1. Install Glunovo Share App

Download the iGO -Share App from Google Play/App store.

App is suitable for the system of Android 6.0, IOS12 and above.

App is compatible with:

- ✓ **Samsung:** Galaxy S8, Galaxy S8 Plus, Galaxy S9, Galaxy S9 Plus;
- ✓ **VIVO:** NEX, X21;
- ✓ **OPPO:** R17, R17Pro, A72;
- ✓ **OnePlus:** 6;
- ✓ **Huawei:** Mate10, Mate10 Pro (and above), P10, P10 Pro (and above), honor 9 series, honor 10 series;
- ✓ **Xiaomi:** 8 series, MIX2 series, Redmi 9A;
- ✓ **iphone** 6 and above.

Note:

Contact your Service Provider if none of these is compatible.



2. Actions to take

- Make sure that your smart phone's Bluetooth is turned on.
- Turn off the Do Not Disturb mode on your smart phone.
- Make sure that IntelliGo App and iGO -Share App are both opened and running.
- Make sure that the iGO -Share App has access to WLAN and mobile data.
- Make sure that the iGO -Share App is always open and running.
- Restart the iGO -Share App after you restart your smart phone.
- Do not activate low battery usage mode while using the iGO -Share App.

1. Register in iGO - Share App

- Enter your name, email, country info and mobile number and tap **REGISTER**.
- Your account is your email. One App pairs with one account only.
- Enter the verification code sent to your email.
- Please set the password with upper or lower letters, numbers, and special characters, at least 8 digits.




Register Account

2. Data transfer

The iGO - Share App is only used for data transfer and display.

Check your last data upload time.

Note: You change your CGM settings and do calibration only on IntelliGo App.

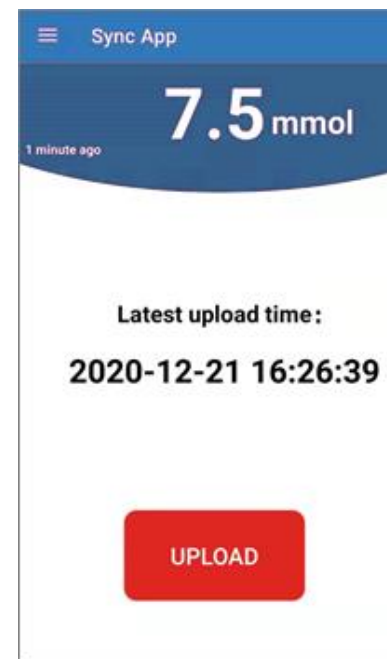

The image shows a login form with the following elements:

- An input field labeled "email account" with a person icon on the left.
- A password input field with a lock icon on the left and masked characters "*****".
- A security code input field containing "3994" with a shield icon on the left. To its right is a grey box containing the handwritten number "3994".
- Two checked checkboxes: "terms of use/privacy policy" and "remember the password".
- A link "forget password?" located to the right of the "remember the password" checkbox.
- A blue "login..." button at the bottom.

Login the webportal using the same account in IGO - Share App
www.intelligofreeportal.com

Refer to user manual for detailed information.